


A271-3 MASSAGE CHAIR

OPERATION INSTRUCTIONS

SAVE THESE INSTRUCTIONS

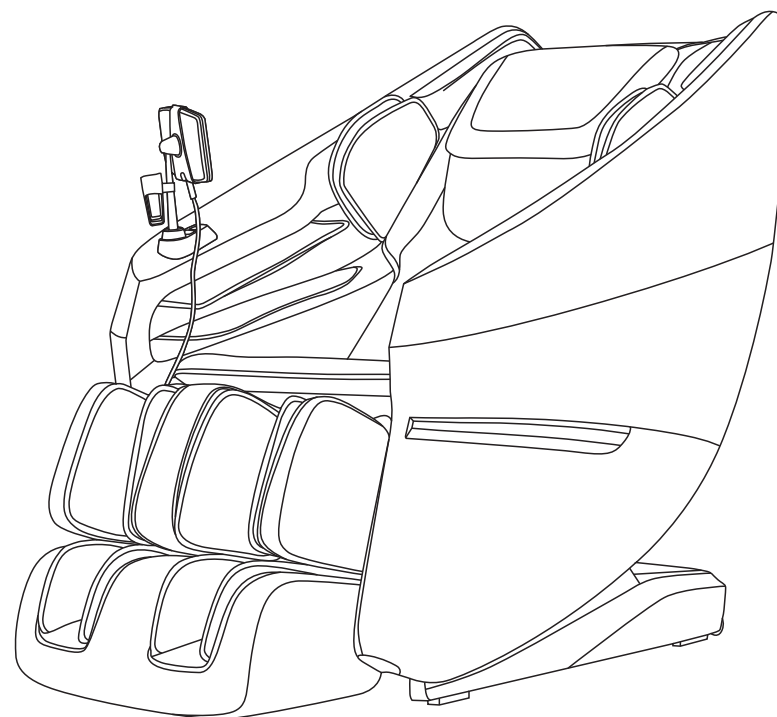

Thanks for purchasing this product. Please read this manual instruction carefully in order to know how to use this product correctly before operation. Please pay more attention to some important safety warning and keep this manual book properly for future use.

Note: Our company reserves the right to change and explain the design of the product. If there is some difference between the product function in market circulation and the description in the manual, it is a normal update of the product, and our company will not inform you otherwise. The specific product function and color shall prevail in kind.

TABLE OF CONTENTS

Before use

Safety Precautions -----	02-05
Installation parts -----	06
Place massage chair -----	07-08
Before use -----	09-10
Product structure -----	11-13

Usage method

Operation Instructions -----	14-19
Massage close -----	19

Maintain



Cleaning and Maintenance -----	20
--------------------------------	----

Important

Troubleshooting -----	21
Product Specifications -----	22

Safety Precautions (Must be observed)


- Please read all relevant instructions carefully before use.
- The notes marked here will help you to use the product safely and correctly.
- In order to clearly indicate the extent of harm and damage, precautions are divided into **"Warning"** and **"Caution"** that may be caused by incorrect use. Both of them are important for safety and must be followed.

 Warning	Indicates that serious injury or death may result from improper use.
 Caution	Indicates that misuse may cause minor injury or property damage.

- Please pay attention to the symbol before each safety item.


 Forbidden	Absolutely Prohibited
--	-----------------------

The following populations should not use this product


 Forbidden	<ul style="list-style-type: none"> ■ This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety. ■ Children should be supervised to ensure that they do not play with the appliance. ■ The appliance has a heated surface. Persons insensitive to heat must be careful when using the appliance. ■ Patients with osteoporosis. ■ Patients with severe heart disease and wearing implantable medical devices such as pacemakers and defibrillators that are susceptible to electromagnetic waves. ■ People who have a fever (above 38 degrees). ■ Pregnant women or shortly after childbirth and during their menstruation period. ■ People who follow medical advice and are prohibited from receiving massage due to thrombosis, aneurysm, severe varicose veins, or other circulatory diseases. Within two weeks after the appearance of blood clots, it is strictly prohibited to massage any form of body parts that have blood clots. ■ People with injuries or skin conditions. ■ Unsupervised children under the age of 14, or anyone unconscious ■ People advised by their doctors to rest or those feeling unwell. ■ People with spinal abnormalities or scoliosis are prohibited from receiving this product due to medical advice due to other diseases. ■ People with wet bodies.
---	---


Safety Precautions (Must be observed)

Important safety warnings


 Warnings	<ul style="list-style-type: none"> ■ If the supply cord is damaged, it must be replaced by the manufacturer, its service agent or similarly qualified persons in order to avoid a hazard. ■ Keep children away from the movable parts of this product. ■ Use a properly grounded power source that is compatible with this device. ■ Unplug the power cord after use or before cleaning to prevent personal injury or damage to the device. ■ Use this device according to the instructions in the manual. ■ Do not use any parts or accessories that are not recommended. ■ Do not use this product outdoors. ■ Do not use near open flames such as stoves, and do not smoke while using. ■ Do not use heating equipment such as electric blankets on this product. ■ Do not sit on the back of the chair, leg massagers, shoulders, armrests, or stand on the seat. ■ Please read the instruction manual before operation. ■ Never use this product for any purpose not listed in the manual. ■ Limit each use to 20 minutes. ■ Do not use this product if the cover or leather is broken or damaged. ■ Do not drop anything into this product. ■ Never fall asleep while using this product. ■ Do not use this product if you are intoxicated or feeling unwell. ■ Do not use this product within one hour of eating. ■ Do not apply excessive force when using this product to avoid injury. ■ Do not use for purposes other than massage. ■ Precautions: Keep away from children. Do not store near high temperatures or open flames. ■ Fully insert the power plug into the outlet and ensure secure contact during use. ■ When using an extension cord, only use nationally certified products. ■ Disconnect power when leaving the premises. ■ Immediately discontinue use and disconnect the power if abnormal noise, odor, overheating, or malfunction occurs. ■ Do not power the unit from lamp socket outlets. ■ Total current draw of connected appliances must not exceed the rated current of the main outlet/circuit. ■ Never use near flammable/explosive materials. ■ Do not disassemble this product or modify its internal wiring, components, or safety features. ■ When unplugging, do not pull on the power cord. ■ Only use specified fuse ratings - never substitute with conductive materials or incorrect ratings. ■ Do not alter the trip settings of circuit breakers or residual current devices.
--	---


Safety Precautions (Must be observed)


Safety precautions	
 Caution	<ul style="list-style-type: none"> ■ Check whether voltage is consistent with the specification of this product. ■ Never insert or unplug the power cord with wet hands. ■ Do not allow water to enter this product to avoid electric shock or damage. ■ Do not pull the power cord when plugging or unplugging. Avoid rough handling. ■ Do not damage the wire or modify the circuit of this product. ■ Do not clean electrical parts, such as the power switch or plug, with a wet cloth. ■ Stay away from this product during a power failure to avoid injury if power is suddenly restored. ■ Stop using this product immediately if it malfunctions, and contact your local supplier. ■ Stop using this product if you feel unwell, and consult your health care practitioner.

Product care and maintenance	
 Caution	<ul style="list-style-type: none"> ■ This product should be maintained by the service center designated by the manufacturer only. No user should disassemble or maintain it without permission. ■ Do not forget to turn off the main power switch after usage. ■ Do not use this product if the power socket is loose. ■ If the product will not be used for an extended period, roll up the power cord and store it in a dry, dust-free environment. ■ Do not store this product at a high temperature or near open flame. Avoid prolonged exposure to direct sunshine. ■ Please clean this product with dry cloth. Never use thinner, benzene or alcohol. ■ The mechanism of this appliance is specially designed and does not require special maintenance. ■ Do not puncture this product with sharp objects. ■ Do not roll or pull this product on any uneven ground. Instead, it should be lifted before moving. ■ Please use this product intermittently. Do not keep it working continuously for a long period of time. ■ Allow intermittent operation - continuous long-term use is prohibited.

Safety Precautions (Must be observed)

Storage conditions	
 Caution	<ul style="list-style-type: none"> ■ Storage room temperature: 5 °C to 40 °C. ■ Relative humidity (RH) between 20% and 80%; free of corrosive gases, and a well-ventilated room.


Operating environment	
 Caution	<ul style="list-style-type: none"> ■ Do not use this product in a highly humid environment, such as a bathroom. ■ Stop using this product immediately when the ambient temperature changes sharply ■ Do not use this product immediately after a sudden change in ambient temperature. ■ Do not use this product in environments with heavy dust or corrosive gases. ■ Do not use this product in confined spaces or areas with poor ventilation.

Solutions to ordinary malfunctions	
 Caution	<ul style="list-style-type: none"> ■ If the motor sounds during operation of this product, this is the normal operation sound. ■ If the controller is not working normally, check if the power plug and socket are securely connected, and ensure the power switch is turned on. ■ If the rated working time is over, this product will automatically activate the power-off button; If this product keeps operating for a long period of time, the temperature protector will stop this product automatically. The product can be used again after half an hour. ■ Weight limit: 120kg. Heavier users should purchase our reinforced models.

Installation parts





Controller bracket installation instructions

After the screw thread on the support knob of the controller rest assembly aligns to the screw thread position of the support mounting seat on the armrest of the massage chair, just turn clockwise and screw tightly for fixation.

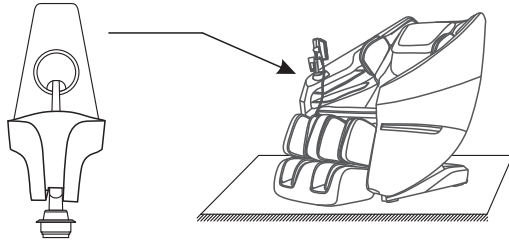

Calf installation instructions

1. After connecting the lower leg and the air tube on the massage chair with the wire harness, untie the fixed pipeline belt and insert the connecting part into the seat frame (Fig. 1).
2. Sleeve the lower leg fixing block on the round tubes on the left and right sides of the lower leg (Fig. 2).
3. Hang the lower leg sleeved with the fixing block on the hook of the massage chair seat frame (Fig. 3).
4. Fasten the lower leg on the massage chair with 4 M4 screws (2 on the left and 2 on the right). Check whether there is any abnormality in the up-and-down turning and whether (Fig. 4).

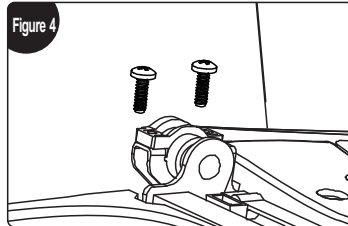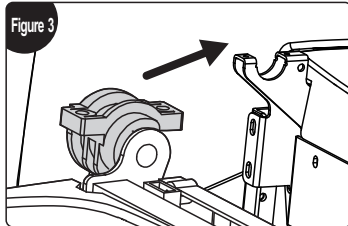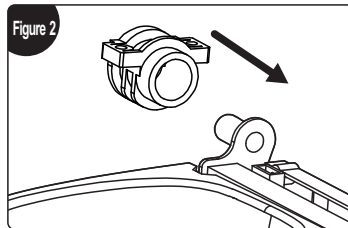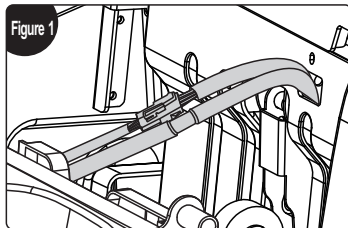


Place massage chair

Installation placement

Make sure there is enough space for the massage chair can be tilted at least 5 cm.



Do not expose the massage chair to direct sunlight or high-temperature environments (e.g., in front of heating devices). Place it in an area where it can avoid heat and direct sunlight. For long-term use in a fixed location, it is recommended to place a soft cushion underneath. The cushion should be large enough to cover the area where the chair and the footrest touch the floor.

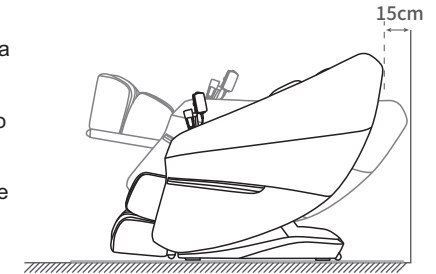


Product lift instruction

Two-person lifting required due to product weight.



1. Please lift the massage chair when moving it on delicate surfaces, such as wooden floors.
2. When moving on similar surfaces, lifting is recommended to avoid damaging the casters. Be cautious when lowering the chair into the designated position to avoid hand or foot injuries. Do not release the chair until it is completely flat. To prevent floor damage, it is recommended to place a mat or protective covering on the floor.

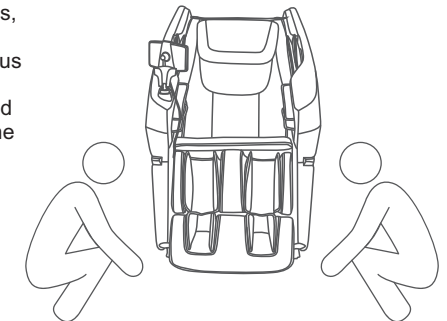


Place Massage Chair

Move method

Use the rear casters to move the massage chair. Tilt the chair backward approximately 45° as shown in the diagram to move it.



NOTE: Excessive force to tilt will cause the chair to tip over completely and product damage.

1. Before moving, ensure the massage chair is properly powered off, with both the backrest and calf sections fully reset, and the power supply disconnected.
2. The chair should remain empty during movement; no people, pets, or objects should be on it.
3. Do not move the chair over obstacles higher than 2mm or gaps wider than 5mm.
4. When moving the chair using the casters, avoid continuous movement over distances greater than 50 meters. If necessary, move the chair slowly and at a steady pace.

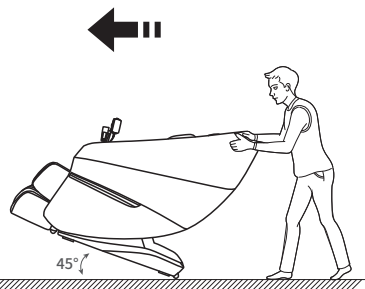

How to disassemble and use headrest cushion and backrest cushion


Using a headrest cushion can reduce the intensity of kneading massage on the neck and shoulders, and you can decide whether to use a headrest cushion according to your needs (recommended to use a headrest cushion). The backrest cushion and the backrest are connected by a zipper (1), and the headrest cushion and the backrest cushion are connected by a zipper (2).

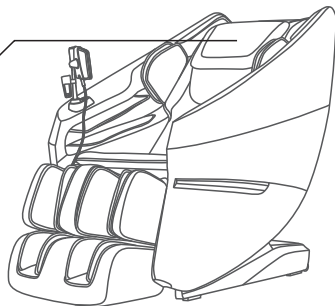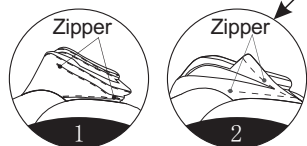

Before Use


Check around

Make sure that there are no people, pets and other objects around the unit.

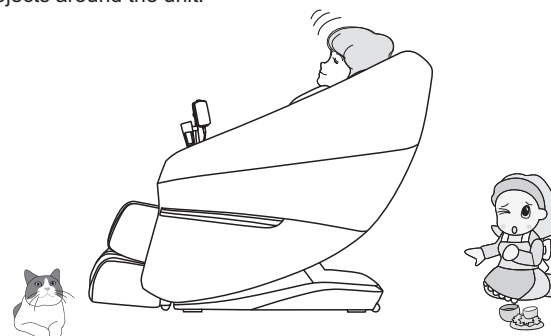

Make sure there are no foreign objects in the gaps of the machine

Before massaging, make sure that no foreign objects are caught in the legs, feet, etc. Under no circumstances allow hands, feet, and head to penetrate into the area between the lower leg mechanism and main body, or the area between the armrest and the capsule. Under no circumstances should hands, feet, or head sink under the fabric or leather covering of the massage area.

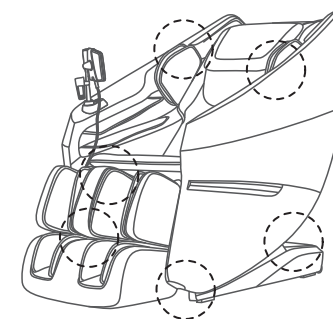


Before Use


Turn on the power

CAUTION


Plug in power cord

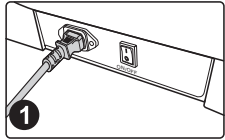

Schematic diagram of power on

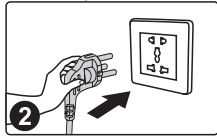

Turn on the power switch ("I" position)

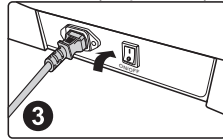


Warnings

- Please check the power cord and plug for damage before use.
- Please supply power to this product in accordance with the working power requirements of this product's calibration.
- It is strictly forbidden to use a power supply other than the nominal power supply of this product.
- It is strictly forbidden to connect various power conversion devices without authorization to supply power to this product.


Product Structure

Parts description

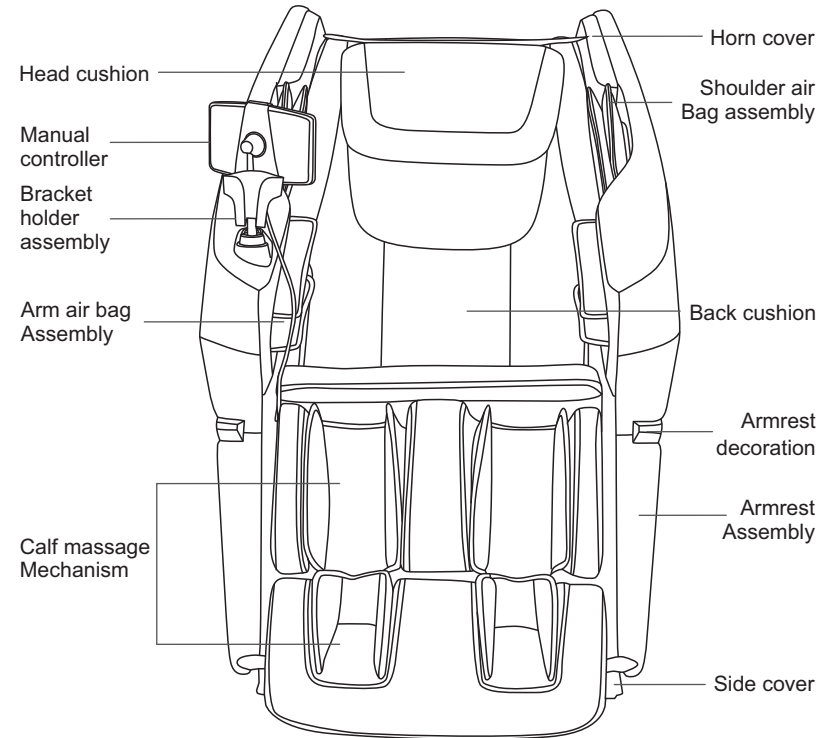

Product Structure


Function distribution

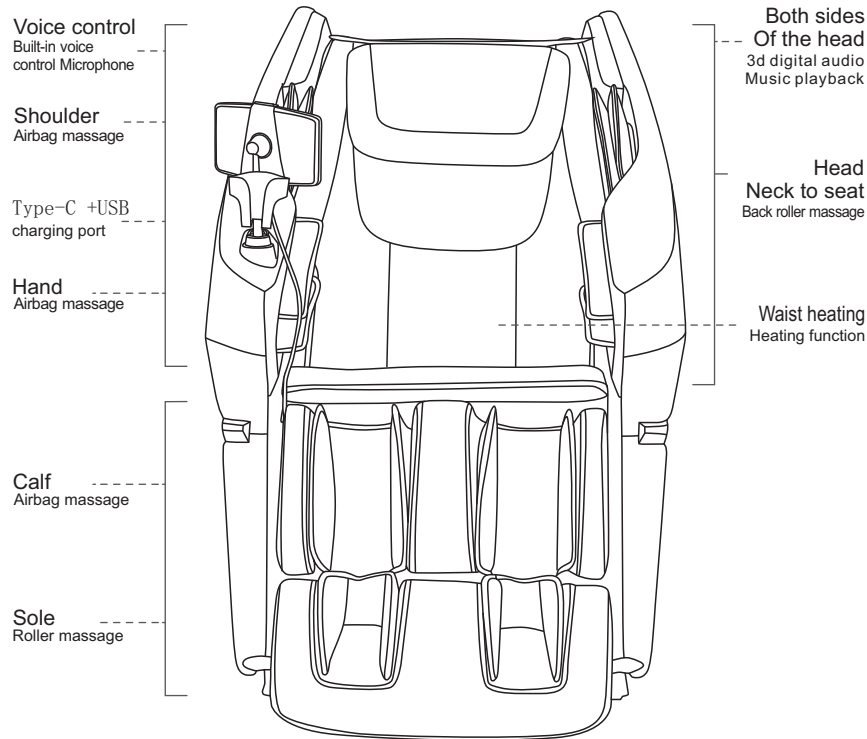


Product Structure

Introduction to the remote controller

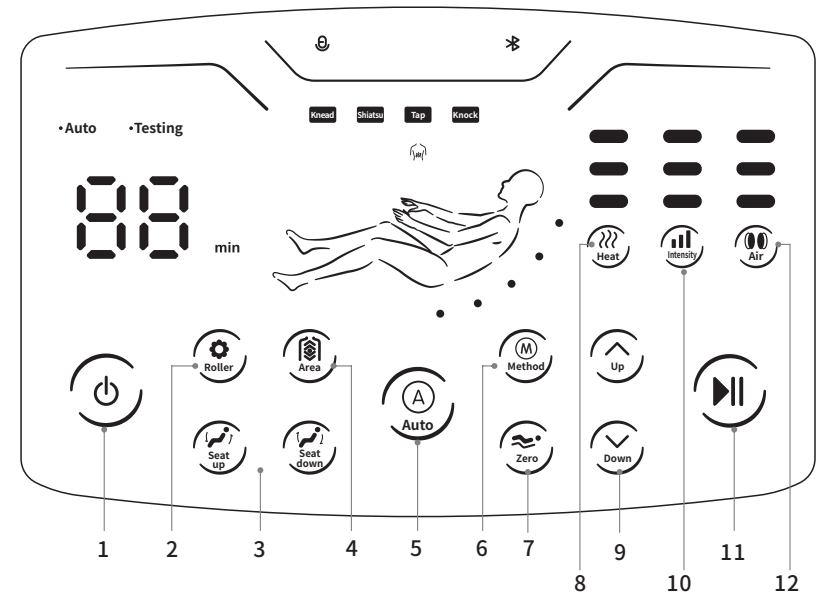

1. When touching the manual controller screen keys in standby mode, the screen will light up.
2. After light up, you can only press the Lift/Lower Leg key and Get Up/Lying Down key to adjust the desired angle.



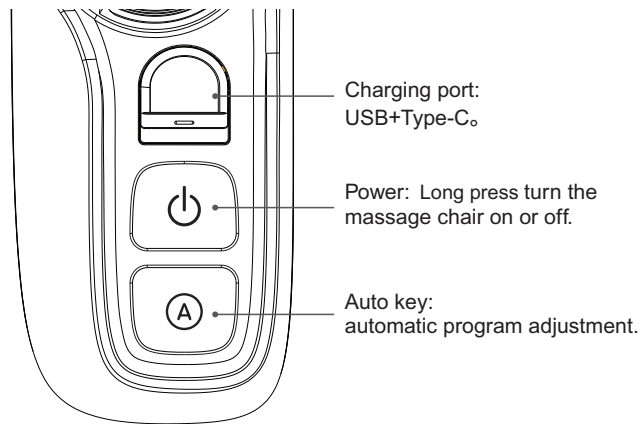

1. Standby key: Turn on or off the massage chair.
2. Foot roller: Short press turn on or off the foot roller massage function. Long press turn on or off the voice control.
3. Seat-up: Adjust the backrest frame up and the calf frame down.
Seat-down: Adjust the backrest frame down and the calf frame up.
4. Massage area: Adjust the massage area.
5. Automatic mode: fast massage, comfort massage, relax massage, full body massage, neck and shoulder massage, back and waist massage, good-night sleep and sport recovery.
6. Massage Method: Short press adjust the massage method. Long press adjust the volume.
7. ZERO: Short press: There are 3 zero-gravity modes and zero-gravity off, which can be cycled and switched as needed. Long press turn on or off the voice control function.
8. Heat: with a total of 3 levels that can be adjusted, or turn off the heating function.
9. Shoulder position adjustment: Manually adjust the position of human shoulder and massage the Shiatsu point.
10. Intensity: massage intensity adjustment, adjustable in 3 levels.
11. Pause: Short press pause or start the massage function. Long press return the massage chair to the sitting angle.
12. Air Intensity: Short press cycle adjustment of air pressure intensity, adjustable in 3 levels, turn off air pressure. Long press adjust the air part.




I, Start massage

- ①.Press the standby key  or long press the right armrest standby key  to start the massage.
- ②.Automatically adjust the angle.
- ③.Automatically detect human shoulder position, keep sitting posture fit with massage chair.
- ④.Conduct fine adjustment of shoulder position. If the detected position deviates from the actual shoulder position, adjust to the proper position by Up and Down keys for shoulder position. Shoulder position is adjustable of 10 gears. If no shoulder position adjustment within 10 seconds, the massage function will be started.



Note: During the detection process, you can choose automatic massage function keys to skip.

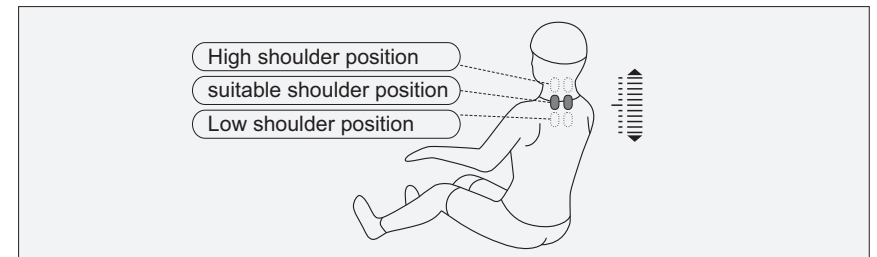Button	Description
 Up	Adjust the massage ball position upward so that it reaches the proper shoulder position.
 Down	Adjust the massage ball position downward so that it reaches the proper shoulder position.




In the process of shoulder position detection, the "beep" sound of the manual controller is heard. At the same time, press the up and down keys of the shoulder position to adjust the shoulder position. At this time, you can also press automatic massage function keys to enter this mode for massage (Step 4 can be skipped directly).

·Pause









Function	Button	Description	Display
Pause		Short press when the massage function is running, press the "Pause" key to temporarily stop all massage functions; press the "Pause" key again to continue the massage function before the pause.	
		Long press return the massage chair to the sitting angle.	/

Operation Instructions




Auto Massage

Function	Button	Description	Display
Auto Massage	 Auto	Fast Massage: Quickly experience all massage effects.	Auto 01
		Comfort Massage: The most comfortable massage experience.	Auto 02
		Relax Massage: Gentle and comfortable deep massage, which can relieve body fatigue and relax the body.	Auto 03
		Full body Massage: Deep massage, which can revive body vitality.	Auto 04
		Neck& shoulder Massage: According to the unique neck and shoulder massage method, through effective massage of cervical spine and scapula, neck and shoulder pain can be relieved and neck and shoulder fatigue can be improved.	Auto 05
		Back&Waist Massage: Focus on the back and waist to relieve back and waist soreness.	Auto 06
		Good-night Sleep: With kneading, shiatsu massage techniques, through the 12 health acupoints massage, help to regulate and balance on human body; so that people can fall asleep more quickly, improve sleep quality.	Auto 07
		Sport Recovery: Quick recovery of body fatigue after exercising.	Auto 08


Manual massage

Function	Button	Description	Display
Massage Method	 Method	Stop	/
		Knead	
		Tap	
		Knead and Tap	
		Shiatsu	
		Knock	
Intensity	 Intensity	Adjustable 3 gears in intensity.	



Operation Instructions

Function	Button	Description	Display
Massage Area	 Area	Fixed point: Massage in a fixed position.	H0
		Partial: Massage in a small area forth and back.	H1
		Upper back: Massage the upper back forth and back.	H2
		Lower back: Massage the lower back forth and back.	H3
		Full: massage the full body forth and back.	H4
	 Up	Massage position adjustment (in Fixed-Point and Partial mode, massage position can be adjusted).	Robot Up
 Down	Robot Down		



Foot roller massage

Function	Button	Description
Foot Roller	 Roller	Short press this key to turn on / off sole roller massage and it is adjustable of three gears in intensity, namely weak, medium and strong, it can be switched cyclically among three gears and off states. Long press turn on or off the voice control.

Heat




Function	Button	Description	Display
Heat	 Heat	With a total of 3 levels that can be adjusted, or turn off the heating function.	

Pneumatic massage


Function	Button	Description	Display
Airbag	 Air	Short press with a total of 3 levels that can be adjusted, or turn off the airbag pressure.	
		Long press adjust the air part. Full body airbag pressure (1), Upper body airbag pressure(2), Lower body airbag pressure(3), turn off the airbag pressure.	1, 2, 3, 0F

Operation Instructions

Pneumatic massage

Function	Button	Description
Seat-up	 Seat up	Press and hold this key, the backrest frame rises slowly, the calf frame descends slowly, accompanying the sound of "beep... beep..." until reaching the top, if the key released the frames stop.
Seat-down	 Seat down	Press and hold this key, the backrest frame moves down slowly, the calf frame moves up slowly, accompanying the sound of "beep... beep..." until reaching the bottom. If the key released the frames stop.
Auto Seat-down	 Zero	Short press every time the key is pressed, it will be switched to next automatic seat-down mode, there are two kinds of automatic modes as well as off mode, which are switched cyclically. Long press turn on or off the bluetooth.

Other

Function	Button	Description
volume	 Method	Long press: the volume can be adjusted up to 6 levels.

Operation Instructions

II. Voice Control Mode

- When the massage chair is turned on or in standby mode, the sound "Hi Alice" or "Hey Alice" will be produced to enable the voice control function.
- After such function is enabled successfully, the massage chair will reply "I am here." At this moment, once a corresponding voice control entry to be implemented is given within six seconds, the corresponding voice control function can be performed after a reply is received from the massage chair; if no correct voice control entry is given within six seconds, the voice control function automatically will be off automatically and needs to be enabled again.
- Note: When waking up the voice control function, please use English, the speech rate is moderate, Voice commands should be follow up the Voice entry.

Voice Control Entry		
Entry: "Hi Alice" or "Hey Alice". Answer: I am here.		
No.	Voice control entry	Answer entry
1	Massage on	Ok, massage on
2	Massage close	Ok, massage close
3	Fast massage	Ok, fast massage
4	Comfort massage	Ok, comfort massage
5	Relax massage	Ok, relax massage
6	Full body massage	Ok, full body massage
7	Neck and shoulder massage	Ok, neck and shoulder massage
8	Back and waist massage	Ok, back and waist massage
9	Good-night Sleep	Ok, good night sleep
10	Sport Recovery	Ok, stretch massage
11	Change the other mode	Ok, change the other mode
12	Fixed massage	Ok, fixed massage
13	Go little down	Ok, go little down
14	Go little up	Ok, go little up
15	Open the air pressure	OK, open the air pressure
16	Close the air pressure	OK, close the air pressure
17	Up the seat position	Ok, up the seat position
18	Down the seat position	Ok, down the seat position
19	Stop adjustment	Ok, stop adjustment
20	Increase the volume	Ok, increase the volume
21	Reduce the volume	Ok, reduce the volume
22	Turn on Bluetooth	OK, Turn on Bluetooth
23	Turn off Bluetooth	OK, Turn off Bluetooth

Remark: Command 13 and 14 can only be used in state of fixed massage. They can use after command 12.

Operation Instructions

III. Operation Instructions for Bluetooth Speaker Play


1. Enter the smart device main menu. Open "Settings" and select the "Bluetooth" option. Open "Bluetooth", and Bluetooth will automatically "Search for a New Bluetooth Device".
2. The smart device will search for the corresponding Bluetooth model (e.g. iRest-XXXX). Click the "Match" button. If you are asked whether to match with the device, you just select "Match" to complete matching.
3. Turn on "Music Player" in the smart device and select the song to play.

IV. Massage close


During the massage, press the hand control standby key "⏻" or long press the right armrest standby key "⏻" to turn off. Will immediately turn off all massage functions, reset the backrest and leg, or when the massage time is up, the massage automatically turns off all massage functions, and the backrest mechanism and leg mechanism do not reset.

Cut off the power supply of the whole machine, as shown in the attached picture (the schematic diagram of the power cut off of the whole machine).


Turn off the power switch ("O" position)

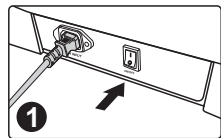

Schematic diagram of unplugging the power plug

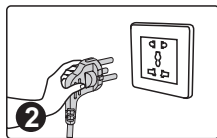

Unplug the product from the power cord

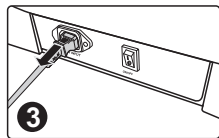

Cleaning And Maintenance

Cleaning synthetic leather and plastic parts

Wipe the leather with a soft, dry cloth.

- Do not use chemicals such as thinner, gasoline, alcohol, etc. to clean the unit.
- When using commercially available leather care products (rags), please follow the instructions for use.
- If synthetic leather is particularly dirty, wipe it in the following way.
 - ① Immerse a soft cloth in water or hot water containing 3% to 5% neutral detergent, then wring it out thoroughly.
 - ② Use the above soft cloth to wipe the leather surface.
 - ③ Wipe off the cleaning solution with a cloth soaked in clean water and thoroughly wrung out.
 - ④ Wipe with a wrung soft cloth.
 - ⑤ Let it air dry naturally.
- When it is difficult to remove stains, please immerse commercially available melamine foam in neutral detergent, and then wipe the machine with it.
- Do not use a hair dryer to dry the surface.
- Be careful when wiping the machine with denim and colored fabrics, as the color of the fabric will stain the surface of the synthetic leather.
- Do not keep these parts in contact with plastic for a long time, as this will cause discoloration.
- Synthetic leather may be discolored, so when using hair dye, please cover the touched part with a towel.



Troubleshooting

Fault	When the calf mechanism or the backrest reaches a certain position, press the corresponding adjustment button and it will no longer change, and a continuous "beep, beep" sound will be emitted.
Solution	Due to product structure design and ergonomic requirements, this product has a limited protection design for the limit state of attitude adjustment. At the same time, a reminder sound will sound when the limit is reached.

Fault	The function does not work or some functions do not work after the machine is turned on.
Solution	After the machine is turned on by pressing the switch key, it is necessary to manually select an automatic massage program or other working state. If it is not operated, the machine will automatically shut down after 20 minutes. If the manual operation mode is selected, the functions that are not selected may be disabled, and the working mode needs to be manually set in the manual mode.

Fault	The machine makes a leather rubbing sound when adjusting its posture.
Solution	This product adopts high-grade anti-corrosion leather that meets environmental protection requirements. Due to the inherent characteristics of leather materials, there will be a normal slight noise when rubbing at the relative movement, which is a normal phenomenon. Note: If you hear fabric or leather tearing sound when the machine is running, please stop using it immediately and seek professional maintenance.

Fault	The massage wheel did not reach the shoulders or neck.
Solution	If the head does not touch the head cushion or the back does not touch the backrest, the shoulder position may be lower than the actual position during the body shape detection process. Sit in the deepest position of the seat, rest your head on the headrest, and start over from the beginning.

Fault	The unit is damaged. The power cord or power plug is abnormally hot.
Solution	To prevent accidents, be sure to contact an authorized service center.

Please feel free to contact the designated service personnel for any questions or doubts about this product.

Product Specification

Model: A271-3

Name: Massage Chair

Rated voltage: 110-120V~ 60Hz
220-240V~ 50Hz/60Hz

Rated power: 90W

Rated time: 20min

Safety structure: Class I